

# Toofypegs®


## Pontefix®

### INSTRUCTIONS FOR USE


TOOFYPEGS® PONTEFIX® IS A TEMPORARY REPAIR KIT FOR BRIDGES, CROWNS AND PINNED TEETH OF ANY MATERIAL. FOR THE PURPOSE OF THESE INSTRUCTIONS THE WORD 'CAP' IS USED IN REFERENCE TO THE CROWN, BRIDGE OR PINNED TOOTH TO BE REPLACED.

- **PONTEFIX® WILL NOT DAMAGE THE CAP, OR OTHER TEETH**
- **PONTEFIX® DOES NOT IMPEDE FURTHER OPERATION BY A DENTIST**
- **THE CONTENTS OF THIS PACKAGING SHOULD ONLY BE USED ONCE**

### BEFORE USING PLEASE READ ALL THE INSTRUCTIONS CAREFULLY

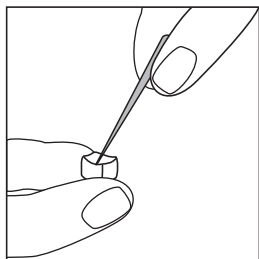


- Scrape off any food residuals from the cap. Clean the cap using a cotton wool bud soaked in alcohol.
- Insert the cotton wool roll provided between the gum and the cheek, near the tooth or pin you want to repair.
- Practice placing the cap over its tooth to ensure you get the correct position.

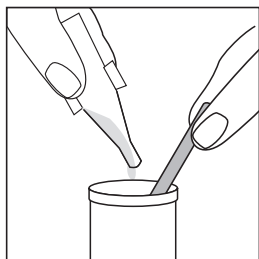


- Remove the lid from the pot of powder. Twist & snap off the top of the liquid bottle provided and squeeze all of the liquid into the pot.
- Mix with the spatula to obtain a smooth, even paste.

#### **WARNING**

**The paste hardens in about two minutes. It is essential to proceed quickly.**

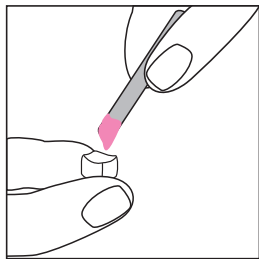

- Using the spatula, half-fill the inside of the cap with the paste.
- In case of a pinned tooth you have to put the paste directly over the pin.

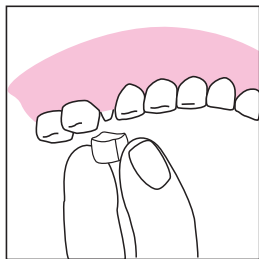

- Position the cap over the tooth.
- Remove the cotton roll, previously inserted between gum and cheek, and put it over the positioned cap.
- Bite down for about 5 minutes.
- When the cap is fixed, remove the cotton roll & any excess paste from around the cap.

THE FIXING OPERATION DOES NOT ELIMINATE THE PROBLEMS THAT CAUSED THE CAP TO COME AWAY. CONSULT YOUR DENTIST IF THE PROBLEM PERSISTS.

#### **KEEP IN A COOL DRY PLACE**

**CE**  
**0373**

Contents:  
1 x cotton wool roll  
1x spatula  
1 x pot of Zinc oxyphosphate (1.2g)  
1 x bottle of Zinc Phosphate solution (0.9ml)

Manufacturer: FIMO srl - via Edolo, 40 20125 MILANO Italy  
Distributor UK: Anglian Pharma Sales & Marketing Ltd  
Titmore Court, Titmore Green, Hitchin, Herts, SG4 7XJ Tel: 01438 743070 Fax: 01438 743080  
Website: [www.anglianpharma.com](http://www.anglianpharma.com) email: [mail@anglianpharma.com](mailto:mail@anglianpharma.com)  
MADE IN ITALY